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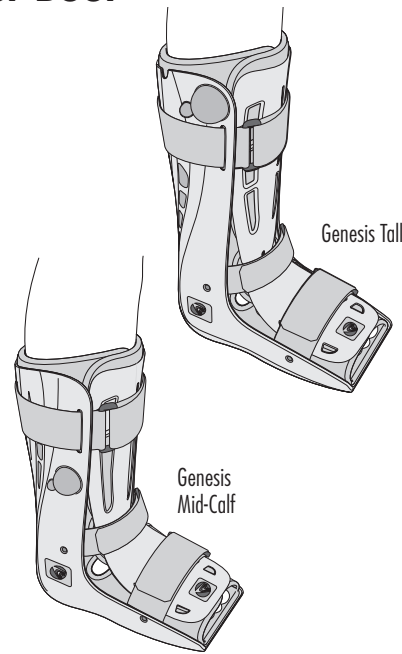
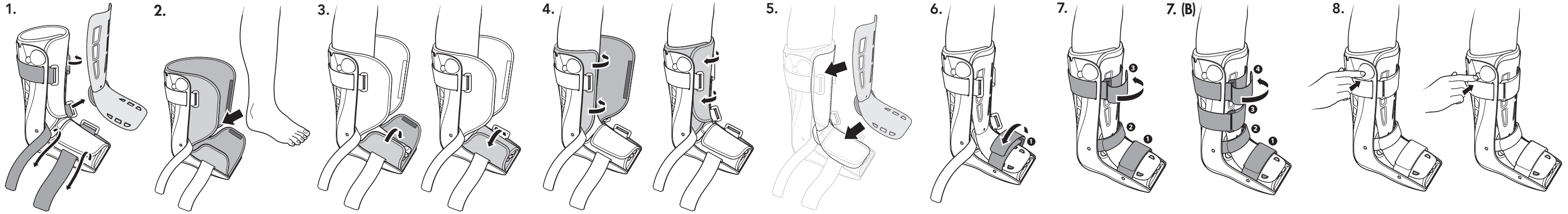
AW-CP020286 Rev D 0915

## English

**INDICATIONS:** For sprains, strains and stable fractures of the lower leg and foot.

### FITTING INSTRUCTIONS:

1. Loosen all of the straps. Detach the plastic anterior plate.
2. Open the foam boot cuffs so that it is easy to insert the foot and calf inside. Place foot into the foam boot so that the heel sits securely against the back of the boot. Make sure the foot is flat on the innersole.
3. Starting with the foot section of the foam boot, fit the boot by overlapping the cuffs. One side has a hook closure that will attach over the opposite side. First, pull the non-hook side of the cuff over the foot, then pull the hook side over and secure when desired fit is achieved.
4. Repeat the process for the calf section of the foam boot.
5. Re-attach the plastic anterior shell. There are hook and loop spots on the back to adhere the shell to the foam boot. Place the shell so the top is aligned with the top of the foam boot.
6. Secure the strap on the top of the foot section (1) first. Insert the strap end through the black D-ring and pull to the desired tension on the strap.
7. Next, secure the bottom calf strap (2). Insert the strap end through the black D-ring and pull to the desired tension on the strap. Repeat the process for the top calf strap (3).
7. (B) On the four strap configuration, secure bottom calf strap (2). Repeat the process for the middle (3) and top calf straps (4).
8. Push the air pump to inflate the air bladder to the desired patient comfort and support. The fitting process is complete.

To deflate the air bladders, press firmly on the release valve until air has been released.

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© 2015 Breg, Inc. All rights reserved.